

**PHOSPHAT
CEMENT**
schnellhärtend
quick setting

CE 0482


MADE IN GERMANY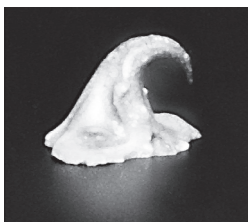

1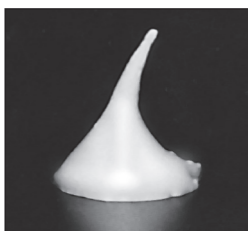

2

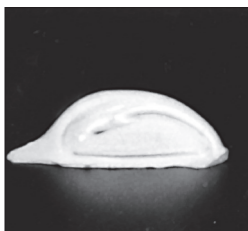3

HOFFMANN'S CEMENT
schnellhärtend
Zinkphosphat-Cement

de

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-10 (grüngrau), -11 (blaugrau), -12 (braun),

-13 (graubraun), -14 (hellrosa), -15 (rosa)

HOFFMANN'S CEMENT
quick setting
Zinc phosphate cement

en

Instruction for use for the dentist

Composition:

Powder contains: zinc oxide, magnesium oxide;

Liquid contains: o-phosphoric acid.

Areas of Application:

Cementation of inlays, onlays, crowns and bridges made of precious metal, non-precious metal as well as metal ceramics (zircon, aluminium and lithium disilicate ceramics) and crowns and bridges on implants, cementation of orthodontic bands, retention pins and screws, cement build up, temporary fillings. Cavity lining as basis for all filling materials (amalgam, composites etc.)

Dispensing Instructions:

Cementation consistency: 1.8 g powder : 1.0 g liquid

Cavity lining consistency: 2.35 g powder : 1.0 g liquid

Dispensing and testing climate for testing in accordance

with DIN EN ISO 9917:

Mix quantities indicated at 23 ± 1°C and a relative humidity of

50 ± 10%.

The dispensing instructions also apply when mixing the liquid

with HOFFMANN'S UNIVERSAL CEMENT.

Processing Recommendations:

A clean, dry glass plate at ca. 20°C (68 °F) and a non-discolouring

spatula are recommended for mixing.

Mixing:

In order to achieve a homogenous consistency, divide the

measured quantity of powder into 4 portions (1/2 1/4 1/8 1/8).

Beginning with the smallest quantity, stroke the powder into the

measured liquid within 90 seconds. The cavity lining

consistency has been reached when on lifting the spatula, the

peak drawn out forms a small hook and does not sink back into

the paste (Fig. 1). The cementation consistency has been

reached when the peak formed on lifting the spatula slowly falls

back into the paste (Fig. 2 and 3). Discard remaining powder or

impure liquid.

Mixing Time: 90 seconds

Processing Time from beginning of mixing (at 23 °C, 72°F):

For cementation: 2:30 minutes

For cavity linings: 2:00 minutes

Setting Time from end of mixing:

For cementation: 3:00 – 5:30 minutes

For cavity linings: 2:30 – 4:00 minutes

Special Information:

• It is vital that opened pulp is covered.

• By deep cavity short-term irritation of the pulp could occur,

however, this irritation will recede completely after a few

minutes. For cavity lining we recommend to use Hoffmann's

Copal Varnish (Ord-Nr. 81901).

• To obtain an optimal pH-value, comply with the mixing time of

90 s.

• The cement should be applied to a preferably dry

tooth surface

• The liquid contains phosphoric acid and is corrosive.

In the event of accidental eye contact,

rinse immediately with large quantities of water.

• Close containers immediately after removing

material.

Package Sizes:

Powder: 100 g, Liquid: 40 ml

Set: 35 g powder and 15 ml liquid

HOFFMANN'S CEMENT quick setting

• Do not use after the expiration date

• Compliant with DIN EN ISO 9917

• Material safety data sheet (MSDS) available at

www.hoffmann-dental.com

*** 15 Colours/Shades:**

-01 (white), -02 (bluish white), -03 (yellowish white),

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