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SPEIKOPLAQUE / SPEIKOPLAQUE 3 % / SPEIKOPLAQUE DUO

Staining liquid for the visualisation of dental plaques

Composition:

SPEIKOPLAQUE and SPEIKOPLAQUE 3 % contain erythrosin (E 127), purified water.
SPEIKOPLAQUE DUO contains Patent Blue dye (E 131), Phloxin B; purified water.

Applications:

Indicator liquid to visualise plaque on dental surfaces. Plaques assume a more or less red colour after application of SPEIKOPLAQUE or SPEIKOPLAQUE 3 %, whereas SPEIKOPLAQUE 3 % colours more intense.

After application of SPEIKOPLAQUE DUO assume older plaque blue colour, younger plaque red colour.

Contraindications:

Allergic reactions to erythrosin, iodine and sunlight (photodermatosis).
Hyperkinetic syndrome in children (SPEIKOPLAQUE and SPEIKOPLAQUE 3 %)
Children under six years of age (SPEIKOPLAQUE and SPEIKOPLAQUE 3 %)

Side-effects / interactions:

unknown

Dosage instructions, mode and duration of application:

Add several drops of the indicator liquid into a Dappen-dish. The liquid is applied to the dental surface with a brush or cotton swab. Already existent plaques become immediately visible by red (SPEIKOPLAQUE DUO also blue) colour. Removal of the stain is accomplished by rinsing with water or teeth-brushing.

Special instructions:

Do not use this medical product after its shelf-life has expired.
Keep out of the reach of children.

Available in the following packs:

liquid	SPEIKOPLAQUE	Contents: 20 ml	Art. No.: 1052
liquid	SPEIKOPLAQUE 3 %	Contents: 20 ml	Art. No.: 1075
liquid	SPEIKOPLAQUE DUO	Contents: 20 ml	Art. No.: 1054

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SPEIKOPLAQUE SPEIKOPLAQUE 3% / SPEIKOPLAQUE DUO

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SPEIKO®- Dr. Speier GmbH P.O.Box 7204 D-48039 Münster
SPEIKO® - Dr. Speier GmbH Postfach 7204 D-48039 Münster